

**DO NOT TILT THE BOX UPSIDE DOWN AND TRY TO USE GRAVITY TO REMOVE THE CHAIR, IT

WILL RESULT IN DAMAGE TO THE CHAIR**

UNBOXING



BOTTOM

- · Open the top of the box as received and remove the foam lid inside the package
- · To remove the power chair, orientate the box such that when it is laid down on the ground the padded bar is at the top and the footplate is at the bottom
- · Grab the padded bar at the top and begin to pull your Travel Buggy out horizontally parallel with the ground.

SETUP







- · Locate the joystick controller. Slide the joystick controller into the arm rest tube (image 1). The length of the joystick controller can be adjusted for comfort before or after secured
- · Tighten the corkscrew to secure the joystick (image
- Align the arrows of the joystick connection cables and press them together to plug the joystick controller in (image 3)
- · Rotate the coupling over the connection point to secure the connection point

GETTING IN AND OUT





- · Before sitting in the Power Chair, be sure the power button is OFF and the rear brake levers are in the DOWN (locked) position
- If you prefer a front entry, we recommend flipping up the foot plate to create an easier sitting experience.
- · DO NOT step on the footplate when entering or exiting CITY 2 PLUS as it may cause the power chair to move in an unsafe manner
- To enter on the side pull on the arm release button as shown in image 1. Raise arm rest back and away. When seated, lower arm rest back into position and ensure it is locked.

UNFOLDING



- · Lay the chair flat on the ground as illustrated above.
- · Hold the handle and pull up the chair while pressing the edge of the seat to open similar to a lawn chair.
- · Continue to unfold the power chair until it is in fully expanded seating position.

QUICK START GUIDE

CITY 2 PLUS Folding Power Chair

FOLDING



- · Press down the seat back lock mechanism to release the chair for folding
- Pressing down on the top of the seat back, begin folding the power chair towards the around
- · The power chair is now completely folded. CITY 2 PLUS can also be folded on to its rear wheels and stands on its own for greater convenience.

CONTROLLER

INCREASE SPEED Press the PLUS + button

DECREASE SPEED Press the MINUS - button

5-Speed Control Lights

- 1-Light On 0.5 MPH
- 2-Lights On 1 MPH

- All 5-Lights On 4 MPH



Power ON/OFF

POWER ON/OFF Press the RED button to turn ON/OFF the power chair

Battery Indicator Lights


There are a total of 5 battery power indicator lights: 1 Green, 3 Yellow and 1 Red. From right to left the lights will disappear as the battery drains indicating the amount of charge remaining. Recharge at any point before the last Red light disappears, indicating a fully depleted battery.

- · CITY 2 PLUS Power Chair has an easy to use 360 degree joystick controller featuring multiple speeds and a battery life indicator
- · Take some time in a safe, controlled environment to learn the operation of the controller, the speed and the maneuverability of the model
- Be sure to adjust the speed depending on your environment and while turning
- · To slow down, slowly bring the joystick back toward the initial position.
- To stop, just bring the joystick back to the initial position
- Use extra care when you drive the power chair in reverse

FREE WHEEL MODE



- To put your CITY 2 PLUS in Free Wheel Mode, come to a full stop and turn OFF the power on the controller.
- Push UP on the two red levers located on each rear wheel as shown.

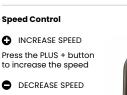

to decrease the speed

SPEED CONTROL LIMITS

3-Lights On 2 MPH

